

Note: Your stool may appear different from the one pictured, but the procedure is the same.


⚠ Caution:

Use approved safety goggles when working on stool.



There is more than one hole pattern in the seat board. Please note placement of screws prior to disassembly.

1. Raise seat to full up position.
2. Place stool upside down on a clean work surface.
3. Firmly grasp the caster base.
4. Remove the (4) screws that secure the seat to the control plate.

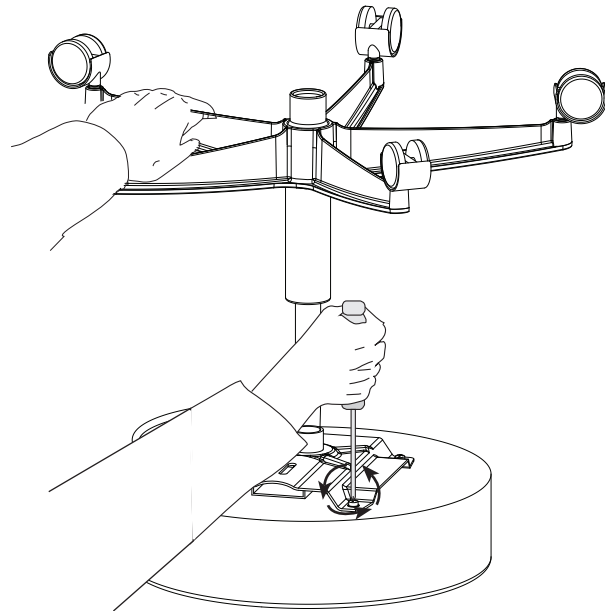

5. Move caster base/cylinder/control plate assembly to new seat.
6. Start by hand, but do not tighten the (4) new screws with the (4) washers.
7. Tighten the (4) screws.
8. Remember to check the tightness of the screws from time to time.

