




(Sample configurations-Not all configurations shown.)


Item No.	Description	Qty
1	5 Star Base	1
2	Glides or Casters-set of 5 (glides standard, casters optional)	1
3	Pneumatic Cylinder	1
4	Seat Assembly	1
5	Footring (only on models ending in -2 or -3)	1

Assembly

1. Remove all contents from the carton. Ensure that you have all components listed above. Do not discard packaging material until satisfied that the stool is fully functional and in good condition.
2. Install glides or casters into the 5 star base by pushing the stems into the holes. Set 5 star base on floor.

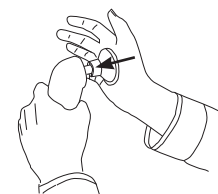


3. Install the footring, if supplied, by turning the footring handknob counterclockwise, and sliding onto the large end of the pneumatic cylinder with the tubular steel ring facing upward. Tighten hand knob.
4. Install the large end of the pneumatic cylinder (with footring if applicable) into base.
5. Place the seat assembly on top of the pneumatic cylinder (small end). Sit on the seat several times to ensure a tight fit.

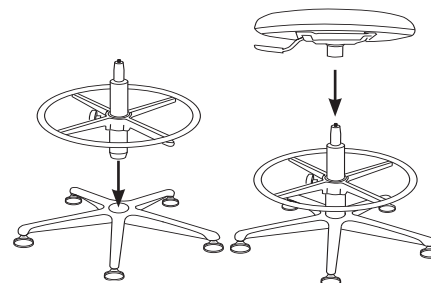

Operation

- **Pneumatic Seat Height Adjustment:** To raise seat, lift weight off stool and lift lever. To lower seat, lift lever while seated.
- **Adjustable Chrome Footring:** Loosen hand knob on the footring by turning it counterclockwise. Raise or lower footring to desired height, and tighten hand knob by turning clockwise.

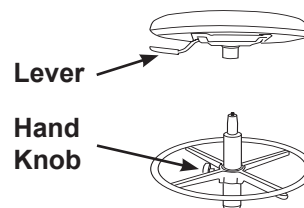

The Brewer Company LLC
13901 Main Street
Menomonee Falls, WI 53051
P 800.558.8934
F 262.251.4391
www.brewercompany.com