

Replacing a Caster Base


Preparing the Caster Base



Caution:

Use approved safety goggles when working on stool.

1. If you are using the casters from the old caster base, simply pull them out to remove them.
2. Install casters into new caster base by pushing all the way in to the sockets at the ends of the legs. Note: if casters still have a fraction of an inch to fully seat, normal use of the stool will usually fully seat them. The casters must be pushed in until locking ring is no longer visible before sitting on stool.

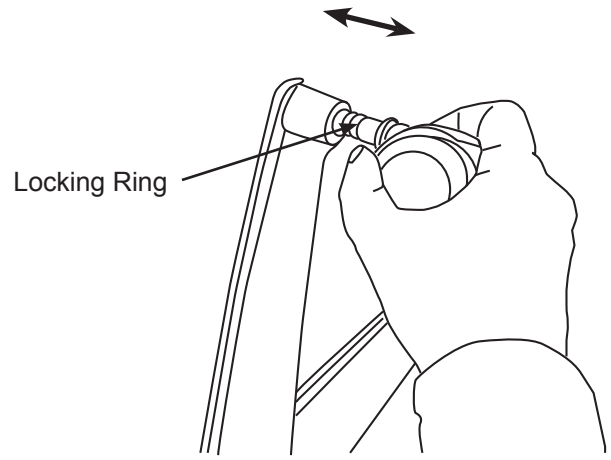

Replacing the Caster Base

Tools: Rubber Mallet

1. Have an assistant hold seat to prevent it from falling when released from caster base.
2. Place protruding portion of cylinder over wooden block and strike caster base with mallet until it releases. Repeated blows with the mallet will be required to cause release. Note: striking the base in different spots around the cylinder will help to ease release.
3. Transfer seat with cylinder assembly to new caster base.
4. Sit on stool to fully seat cylinder into caster base.

