



Model 1500 Blood Drawing Chair

Assembly and Operation Manual



<i>Important Information</i>	<i>2</i>
<i>Safety Information</i>	<i>2</i>
<i>Specifications</i>	<i>3</i>
<i>Assembly.....</i>	<i>3</i>

<i>Operation</i>	<i>5</i>
<i>Maintenance.....</i>	<i>6</i>
<i>Accessories.....</i>	<i>7</i>
<i>Limited Warranty</i>	<i>7</i>

The Brewer Company, LLC
N88 W13901 Main Street, Suite 100
Menomonee Falls, WI 53051 USA



IMPORTANT INFORMATION

General

The Brewer Blood Drawing Chair is designed to provide positioning and support of patients during the blood drawing process conducted by qualified medical professionals. The chair is constructed of tubular steel, welded for strength and longevity.

Read and understand all operating instructions, safety information, and maintenance requirements contained in this manual prior to using the chair. Become familiar with all of the chair functions before using it with a patient.

Service

If you require assistance with the installation or operation of your Brewer Blood Drawing Chair, call the Brewer Customer Service Department at **1-800-558-8777**.

Our trained staff will attempt to assist you in correcting the problem directly over the phone. If service is required, a factory authorized technician will be sent to your location. Please fill in the following information for use when calling the Brewer Company or your distributor with questions regarding your unit. See Figure 1 for model and serial number location.

Date of Purchase _____

Model Number _____

Authorized Dealer Name _____

Dealer Phone Number _____

Dealer Address _____

SAFETY INFORMATION

Safety Information

The primary concern of The Brewer Company is that the equipment is operated and maintained with the safety of the patient and healthcare staff in mind. To ensure safe and reliable operation:

- Read and understand all instructions in this manual before attempting to install or operate the chair.
- Ensure that appropriate personnel are informed on the manual contents. This is the responsibility of the purchaser.
- Ensure that this manual is located near the chair, or if possible permanently affixed to the chair.



This is the safety alert symbol. It is used to alert you to potential physical injury hazards. Obey all safety messages that follow this symbol to avoid possible injury or death.



DANGER Indicates a hazardous situation that, if not avoided, will result in death or serious injury.



WARNING indicates a hazardous situation which, if not avoided, could result in death or serious injury.




CAUTION indicates a hazardous situation which, if not avoided, could result in minor or moderate injury.

SPECIFICATIONS

Weight of Chair.....	44 lbs.	Hip Clearance.....	20 in.
(With Carton)	52 lbs.	Seat Cushion Dimensions	18 in. x 18 in.
Chair Weight Capacity	350 lbs.	Back Cushion Dimensions.....	18 in. x 12.5 in.
Chair Height (Overall)	37 in.		
Chair Depth (Overall).....	23.5 in.		
Width:			
With Both Worksurfaces attached	43.25 in.		
Armrests Only.....	29.25 in.		
Inside Arm Width	19.25 in.		
Seat Height.....	22.0 in.		

ASSEMBLY

NOTICE

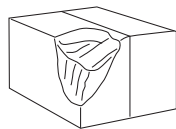

Inspect Carton and Contents

Inspect all boxes and contents for damage. Report any damage to the carrier immediately.

NOTICE



No Sharp Tools

To avoid damaging the chair's upholstery or painted surfaces, DO NOT use a knife or other sharp object to open the packaging.

Overview

Perform the following sequence when setting up the chair.

1. Unpack the Chair.
2. Assemble the Chair.
3. Position Chair in desired location.

NOTICE


Chair weighs approximately 44 lbs.




Chair shown in packed configuration. Armrests are reversed for shipping

ASSEMBLY (continued)

1. Remove Armrests from Chair and set aside. (The Armrests are installed at the factory in a shipping configuration that is opposite of their intended position). See figure on previous page.
2. Insert (4) 1/4-20 x 1.75" screws and washers through seat supports from underneath chair into Seat Bottom and tighten. Seat Bottom should be positioned with Notch and Label facing the front of the chair. (See inset picture to locate front of Seat Bottom)
3. Insert (4) 1/4-20 x 0.75" screws and washers through Tabs on upper frame, from back of chair into Seat Back and tighten.
4. Position Armrests back into tubing, install Hand Knob and tighten.

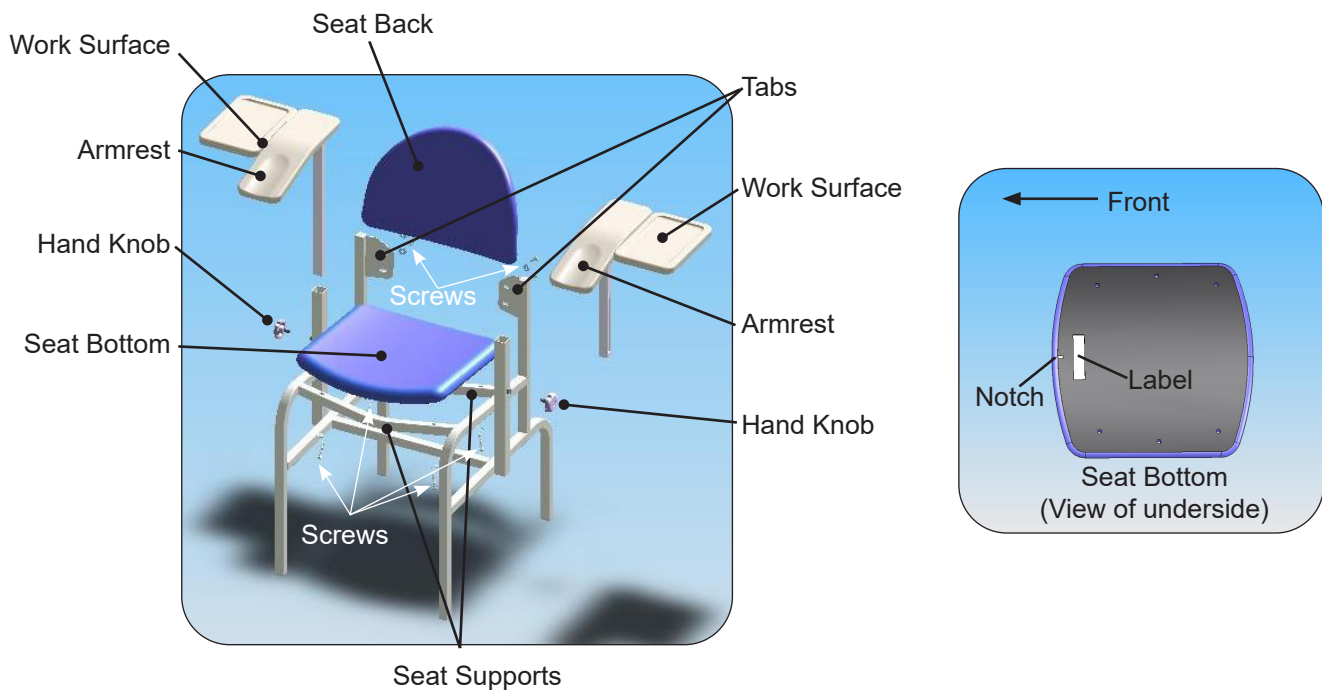

Work Surface Removal: To remove the Work Surface portion of the Armrest, remove (4) screws, slide Work Surface and its support plate away from Armrest, then reinstall the (4) screws into the Armrest.

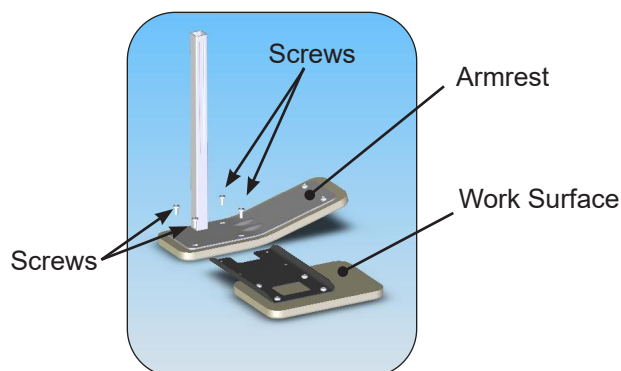

OPERATION

Adjusting the Armrest


1. While supporting the Armrest to be adjusted, loosen the Hand Knob.
2. Move the Armrest vertically to the desired position and tighten Hand Knob.

NOTICE

Make Sure Knobs are Tight

Tighten knobs sufficiently to prevent armrest from lowering while in use.

3. To remove the Armrest completely from the chair loosen the Hand Knob until the armrest lifts freely out of the square tube.

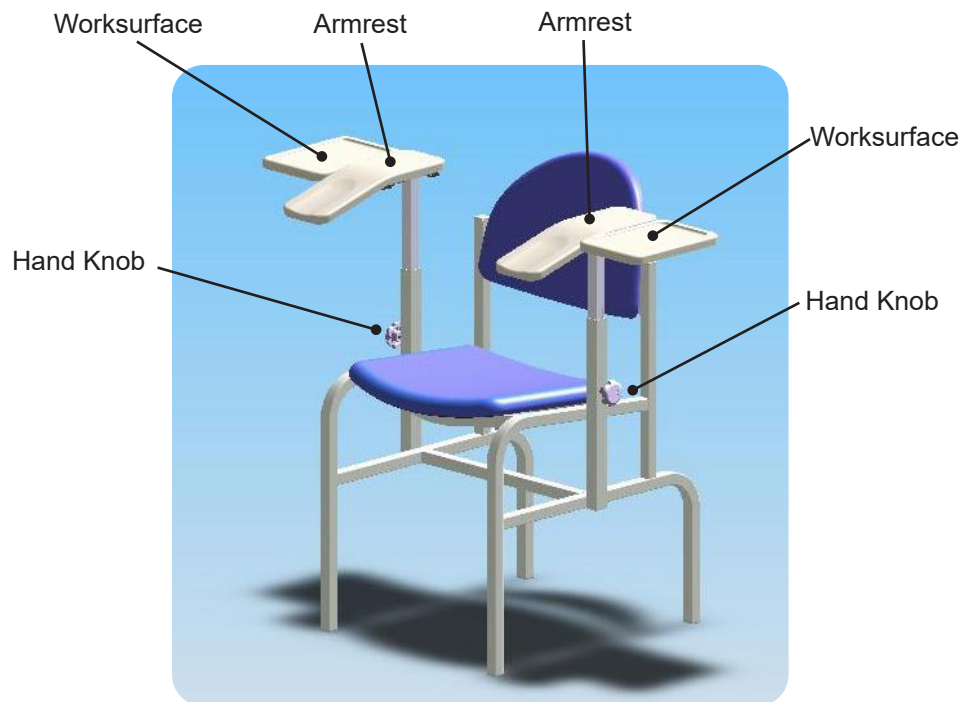

MAINTENANCE

Preventative Maintenance

NOTICE

Loose Fasteners and Damaged Components

Failure to perform periodic inspections of the chair could result in equipment damage.

Inspect chair every 6 months.

- Inspect the mechanical functions to ensure satisfactory operation.
- Check fasteners to make sure they are present and tightened securely.

NOTICE

Table Malfunction

If chair malfunctions, assist the patient from the chair.
Service the chair prior to use.

Table Care

NOTICE

Wipe Spills Immediately!

The upholstery material is resistant to most medicinal type stains, but may be damaged by solvents and dyes. Remove any spilled fluids from the upholstery immediately.

For the latest in Cleaning Guidelines visit:
www.brewercompany.com

ACCESSORIES

Description	Order No.	Use/Restrictions
Drawer Option	101059	Intended Use - Optional storage drawer for blood drawing supplies.

WARRANTY

For Brewer Warranty Information:

Visit www.brewercompany.com or contact the Brewer Customer Service Department at 1-800-558-8777.