

## INSTALLATION

### Overview

Perform the following sequence in order when setting up the table:

- Uncrating
- Leveling the Base
- Installing the Foot Control
- Installing the Upholstery

### Uncrating

*NOTE: Inspect all boxes and contents for damage. Report any damage to the carrier immediately.*

#### **CAUTION**


To avoid damaging the table's upholstery or painted surfaces, **DO NOT** use a knife or other sharp object to open the packaging. Also, to avoid damaging the table lift only at points indicated in Figure 3. Do not lift at other points as indicated in Figures 3 and 4.

1. Using a 1/2" open-end wrench, or socket, unbolt the table from the wooden shipping skid by removing the four bolts located underneath the table's base; two on each side.
2. Remove the table from the shipping skid by sliding it off the front of the skid. Be sure to lift at the correct points as indicated in Figure 3. Position the table in the desired room location.


#### **WARNING**

The table weighs approximately 550 lbs. Two or more people should assist in removing the table from the shipping skid. Also, use proper lifting techniques when lifting. Failure to do so could result in serious injury.

3. Remove packing tape from stirrups (Figure 4). Pull out treatment pan slide and remove packing tape from the treatment pan assembly components. Return treatment pan slide to its stowed position.
4. The electrical rating for this unit is 230 VAC, 60/50 Hz, 3.0 amps. The power cord must be plugged into a matching 230 VAC receptacle.

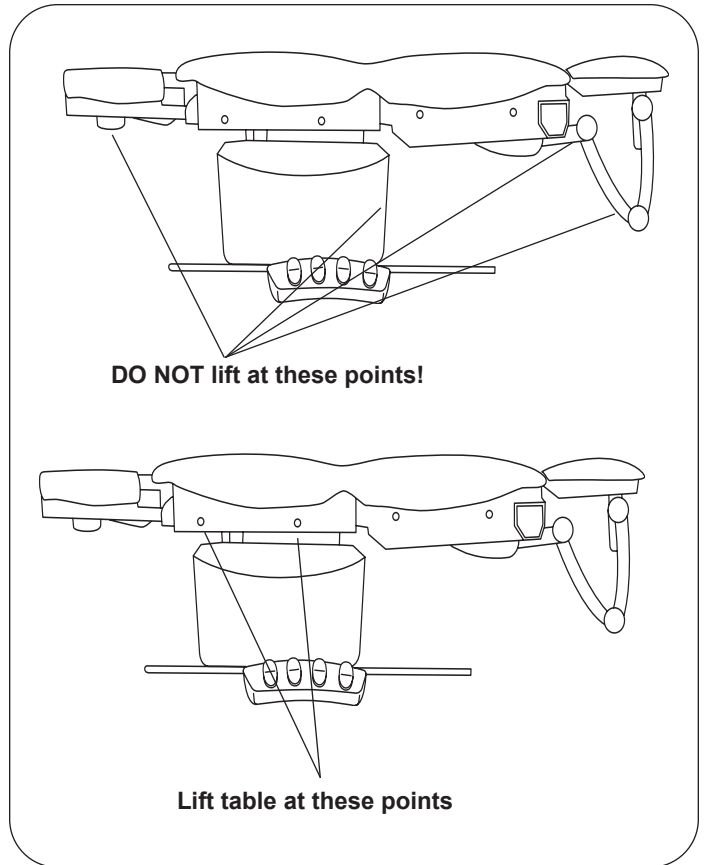

**Figure 3. Table Lift Points**

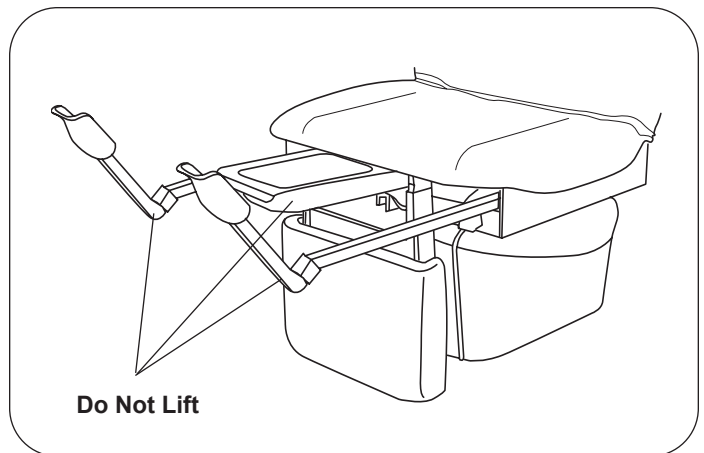


**Figure 4. Stirrups and Treatment Pan**

## INSTALLATION (CONTINUED)

### Leveling the Table

A leveling screw pad (Figure 5) is located in six places under the table's base. Adjust the six leveling pads to achieve a solid, level installation.

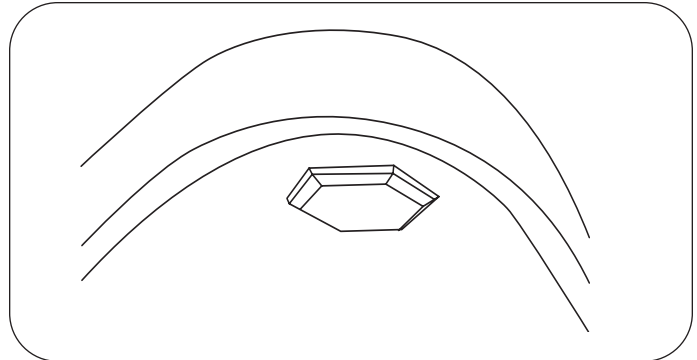


*Figure 5. Leveling Screw Pad (located under table)*

### Installing the Foot Control

1. Remove the foot control from the shipping box.
2. Insert the end of the cord into the foot control receptacle on the table (Figure 6). Make sure the tab in the cord receptacle is pointing upward and receptacle is fully seated. Test the foot control for operation.

### Installing the Power Cord

1. The Power Cord is sold separately.
2. Insert the female plug end of the power cord into the receptacle on the table (Figure 7). Make sure the tabs are aligned and the plug is fully seated.

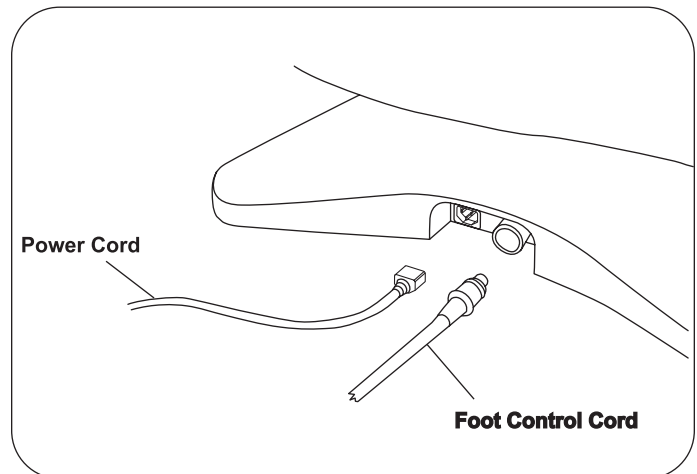

*Figure 6. Power Cord & Foot Control Cord Receptacles*