

INSTALLATION

Overview

Perform the following sequence in order when setting up the table:

- Uncrating
- Leveling the Base
- Installing the Foot Control
- Installing the Upholstery

Uncrating

NOTE: Inspect all boxes and contents for damage. Report any damage to the carrier immediately.

CAUTION

To avoid damaging the table's upholstery or painted surfaces, **DO NOT** use a knife or other sharp object to open the packaging. Also, to avoid damaging the table lift only at points indicated in Figure 3. Do not lift at other points as indicated in Figures 3 and 4.

1. Using a 1/2" open-end wrench, or socket, unbolt the table from the wooden shipping skid by removing the four bolts located underneath the table's base; two on each side.
2. Remove the table from the shipping skid by sliding it off the front of the skid. Be sure to lift at the correct points as indicated in Figure 3. Position the table in the desired room location.

WARNING

The table weighs approximately 550 lbs. Two or more people should assist in removing the table from the shipping skid. Also, use proper lifting techniques when lifting. Failure to do so could result in serious injury.

3. Remove packing tape from stirrups (Figure 4). Pull out treatment pan slide and remove packing tape from the treatment pan assembly components. Return treatment pan slide to its stowed position.
4. The electrical rating for this unit is 115 VAC, 60 Hz, 5.8 amps. The three-pronged grounding plug on the table power cord must be plugged into a matching three-pronged, grounded, non-isolated, correctly polarized 115 VAC receptacle.

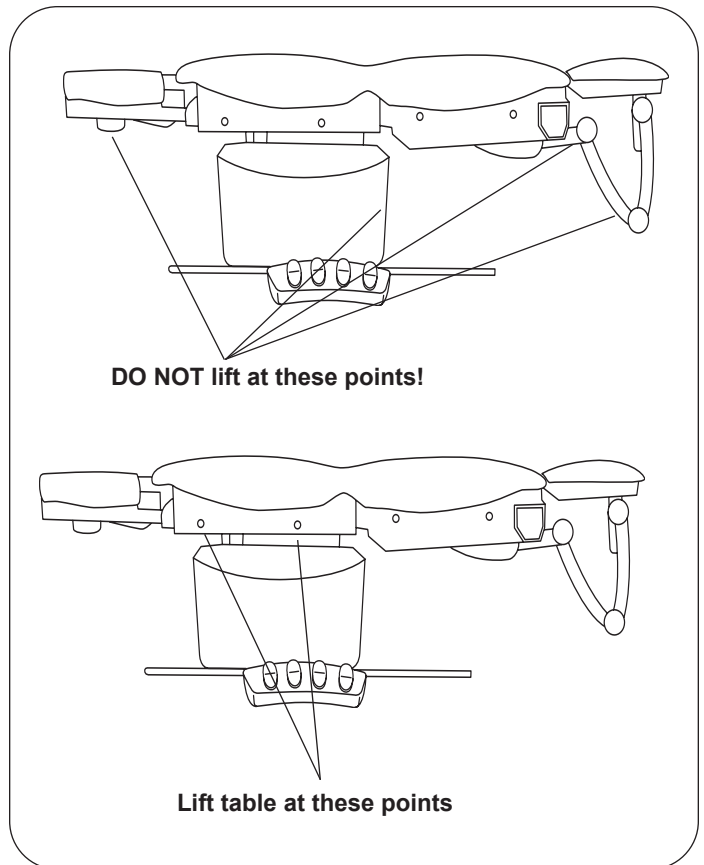


Figure 3. Table Lift Points

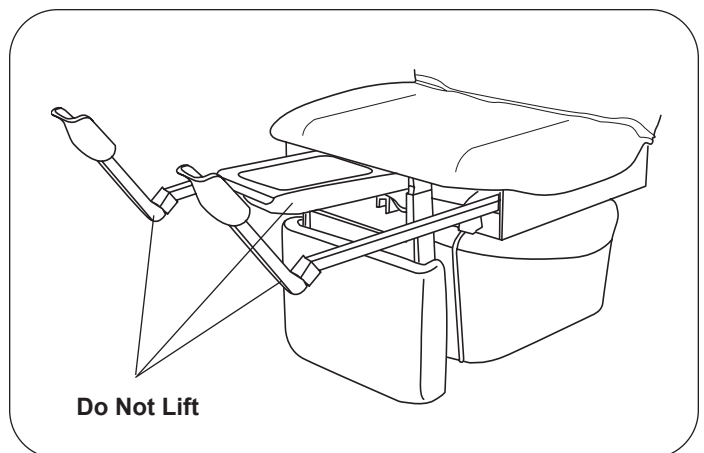


Figure 4. Stirrups and Treatment Pan

INSTALLATION (CONTINUED)

Leveling the Table

A leveling screw pad (Figure 5) is located in six places under the table's base. Adjust the six leveling pads to achieve a solid, level installation.

Installing the Foot Control

1. Remove the foot control from the shipping box.
2. Insert the end of the cord into the foot control receptacle on the table (Figure 6). Make sure the tab in the cord receptacle is pointing upward and receptacle is fully seated. Test the foot control for operation.

Installing the Upholstery

The upholstery set includes: Headrest shroud and cushion, Seat/Back cushion, and Legrest cushion.

HEADREST SHROUD AND CUSHION

1. Raise the back so the top of the headrest is at a comfortable work height.
2. Align the holes in the headrest cushion, shroud, and headrest mounting link. Make sure cushion is properly centered on the shroud. Fasten with the four screws provided.

SEAT/BACK CUSHION

1. Refer to the *Operation* section in this manual and position the table so the seat, back and legrest are flat (in the horizontal position). (Figure 7)
2. Place the seat/back cushion on the table with the curved end toward the headrest and the straight end facing the legrest (Figure 8). Adjust the cushion so that the straight end overhangs the seat frame by approximately 4".
3. Push down lightly in the middle of the cushion and shift it toward the headrest until the 4 posts on the bottom side of the cushion drop into the 4 slots in the seat frame. Pull up slightly on the middle of the cushion to make sure the posts have engaged with the slots.
4. Align the two threaded holes on the headrest end of the cushion with the two holes in the frame. Attach with two screws provided.
5. Align the two threaded holes on the legrest end of the cushion with the two holes in the frame. Attach with two screws provided.

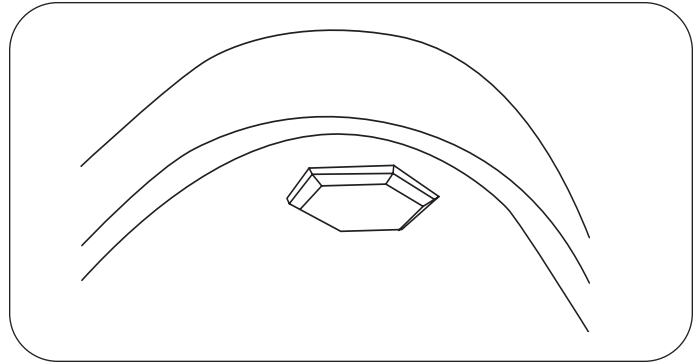


Figure 5. Leveling Screw Pad (located under table)

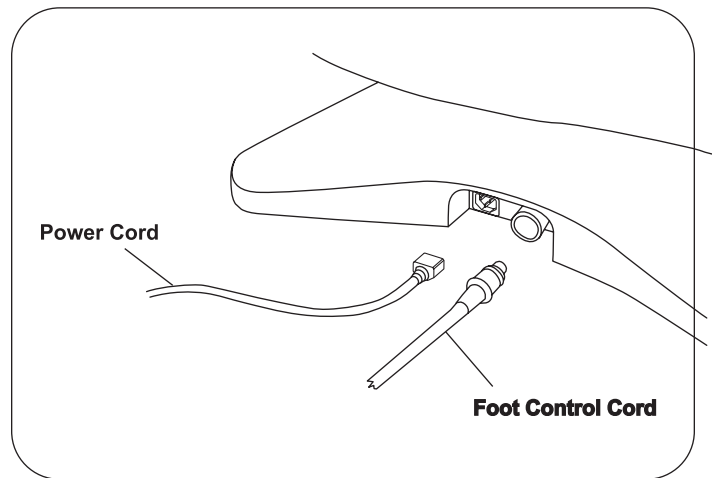


Figure 6. Power Cord & Foot Control Cord Receptacles

LEGREST CUSHION

1. Position legrest in the horizontal position. Raise the table to it's fullest height.
2. Align the holes in the legrest cushion and legrest plate, then fasten with the four screws provided.