


3500162 Toe Board Service Kit

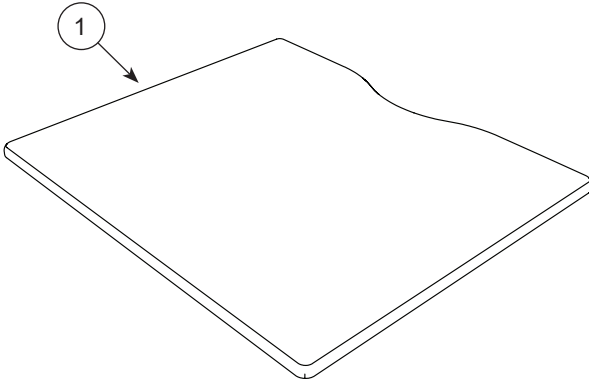


Item No.	Description	Qty
1	Toe Board	1

Registering your Brewer product is easy.
Simply go to brewercompany.com/warranty-registration, fill out the information fields and click send.
By registering your product today, you are better safeguarding your investment in the future.


The Brewer Company, LLC
N88 W13901 Main Street, Suite 100
Menomonee Falls, WI 53051
P 800.558.8777
F 262.251.2332
E customerservice@brewercompany.com
www.brewercompany.com

Remove the Toe Board

1. Pull the leg extension all the way out.

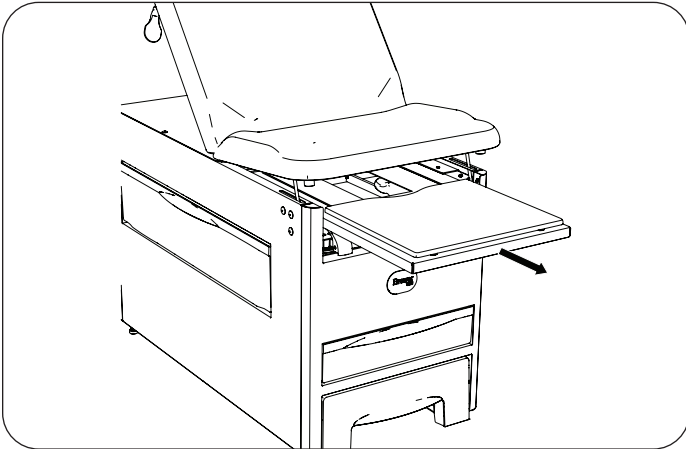


2. Remove the toe board from the leg extension and discard.

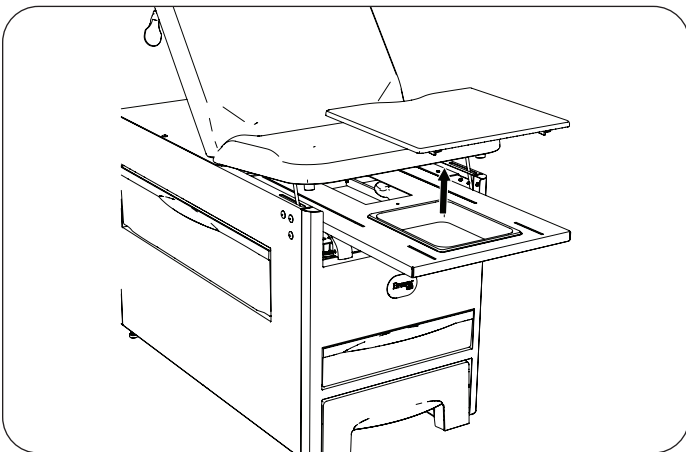


Install the Toe Board

1. Place the toe board into the leg extension.

NOTE: Ensure the curved edge of the board faces toward the rear of the exam table.

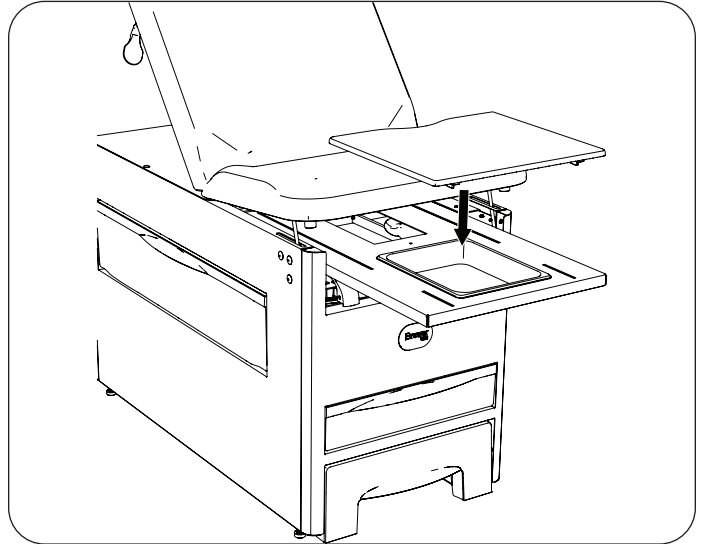

2. Move the toe board back and forth to ensure the board is fully seated.

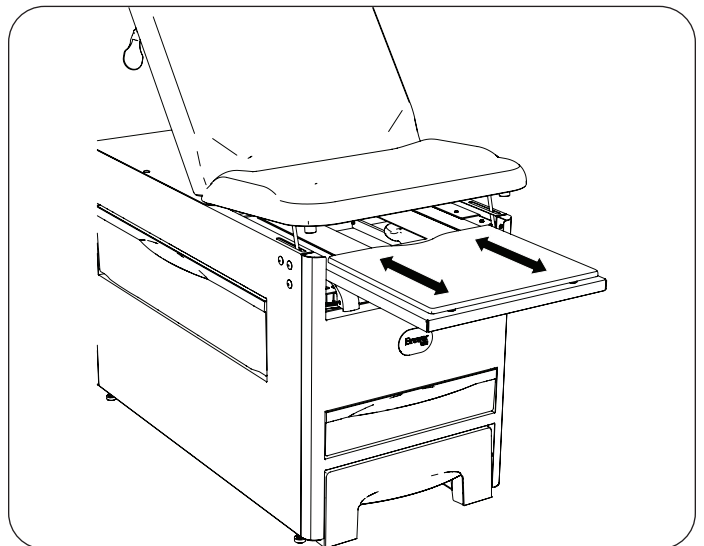

3. Push the leg extension back into the table.