


3500449 Mounting Plate Service Kit

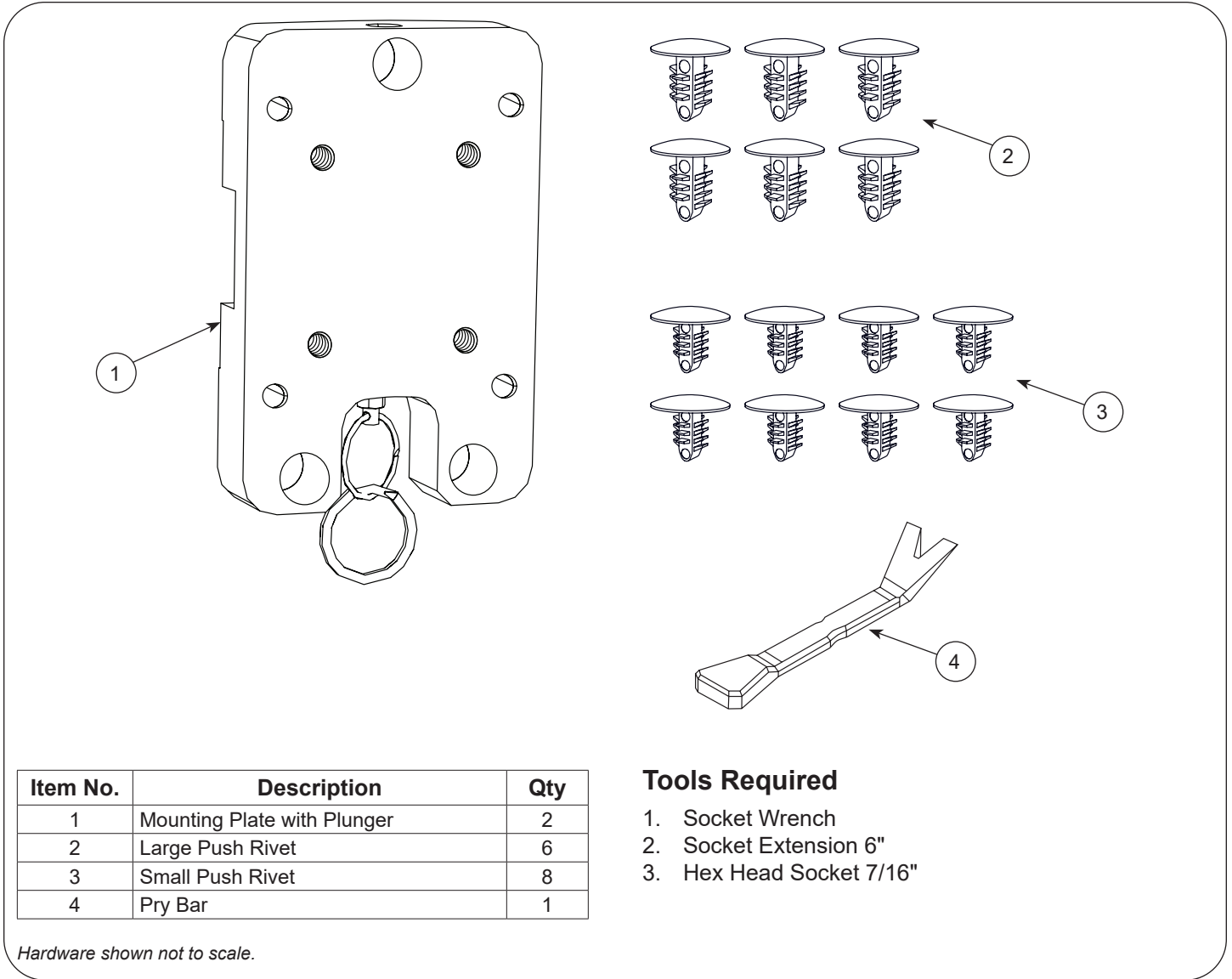

Registering your Brewer product is easy.

Simply go to brewercompany.com/warranty-registration, fill out the information fields and click send.

By registering your product today, you are better safeguarding your investment in the future.

The Brewer Company, LLC

N88 W13901 Main Street, Suite 100

Menomonee Falls, WI 53051

P 800.558.8777


F 262.251.2332

E customerservice@brewercompany.com


www.brewercompany.com

With Knee Crutch Only


1. Remove knee crutch.

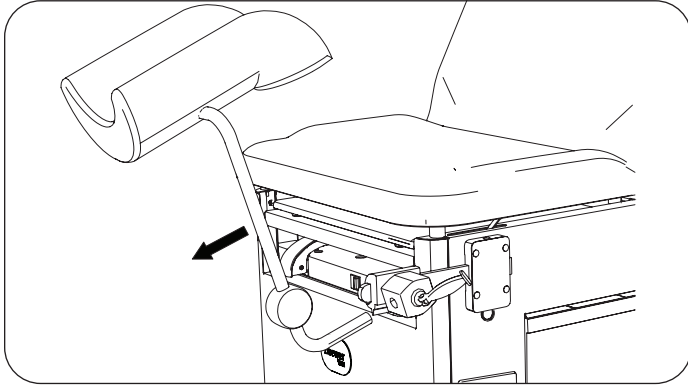


2. Remove knee crutch lock socket.

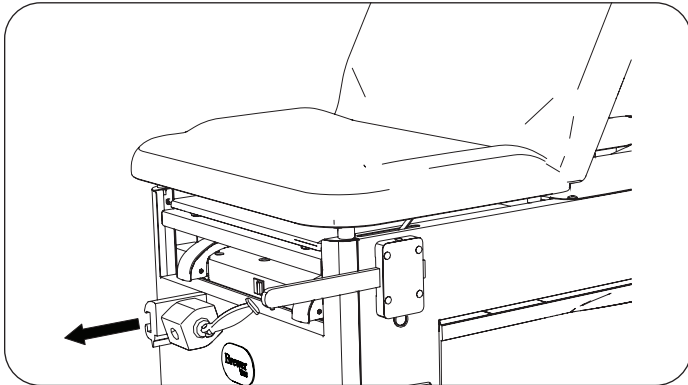


3. Using supplied pry bar, remove four small push rivets attaching mounting plate cover to mounting plate.

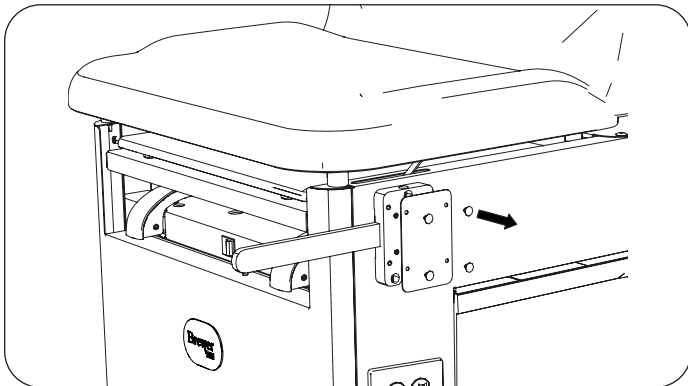


4. To gain access to inside of cabinet, pull out stirrups (if equipped) and leg extension. Remove upholstered pad and debris pan.

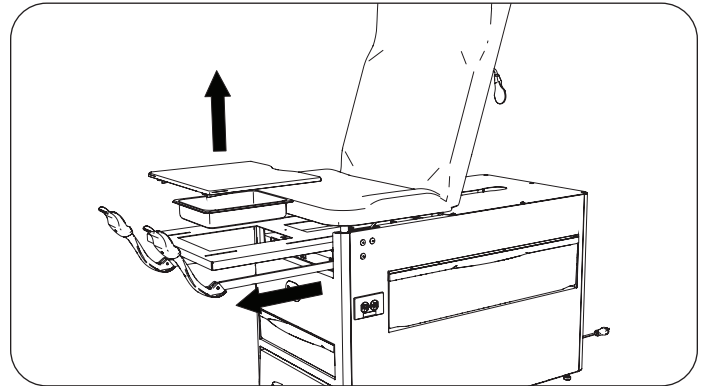


5. From inside of cabinet loosen three hex head bolts attaching mounting plate. Remove mounting plate and beam. Pull plunger downward and remove beam from mounting plate. Discard mounting plate with plunger.

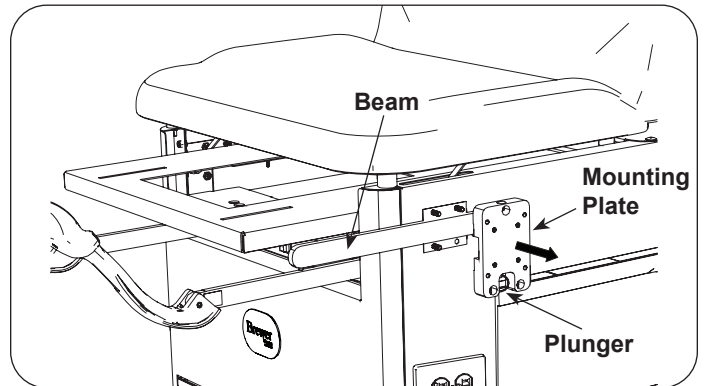


6. Pull plunger downward and install beam in new mounting plate. Round end of beam faces forward and set screws face downward. Position mounting plate and thread/tighten three hex head screws into mounting plate.

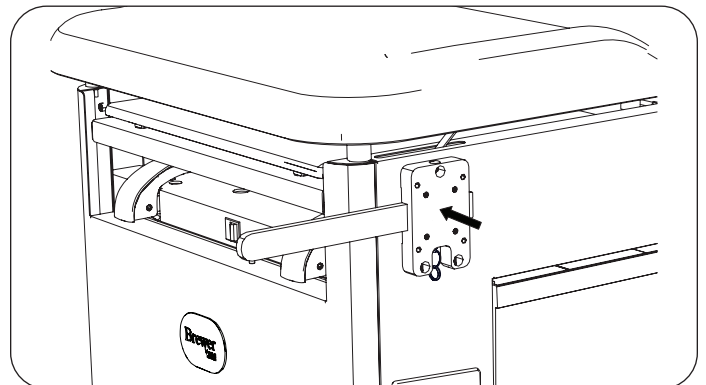


- Attach mounting plate cover to mounting plate using four new small push rivets.

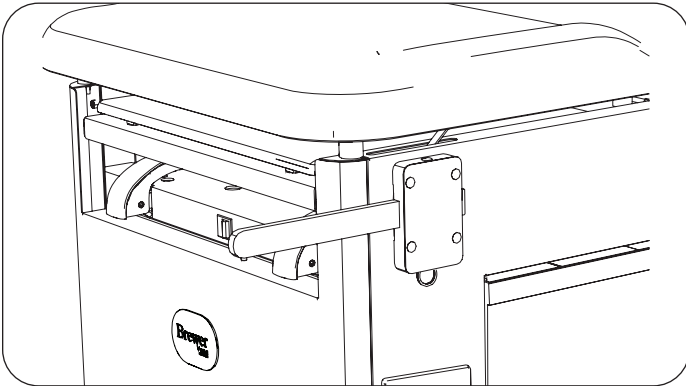

- Attach lock socket to beam.

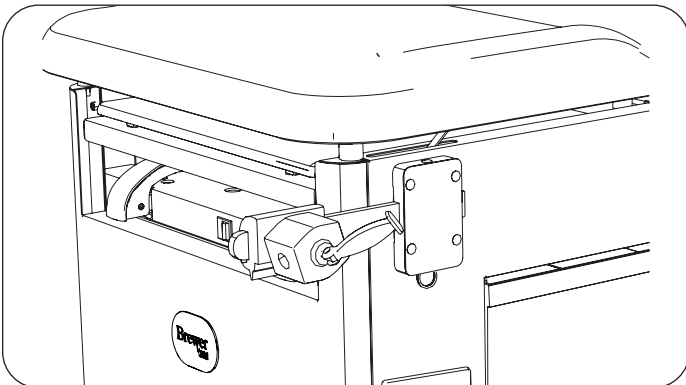


- Assemble knee crutch to lock socket.

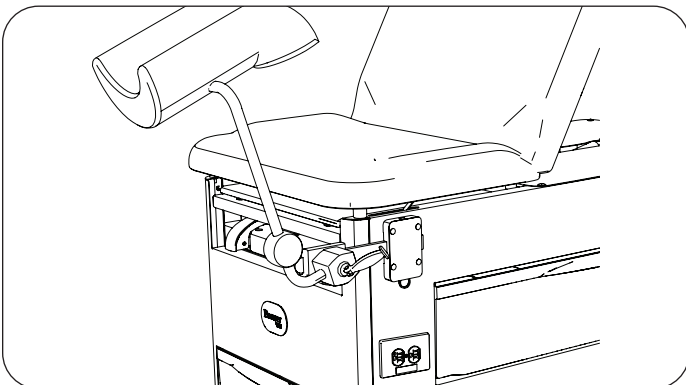


- Repeat for opposite side of table.
- Install upholstered pad and debris pan. Push stirrups (if equipped) and leg extension back into table.

With Assist Handles Only


- Remove assist handle.

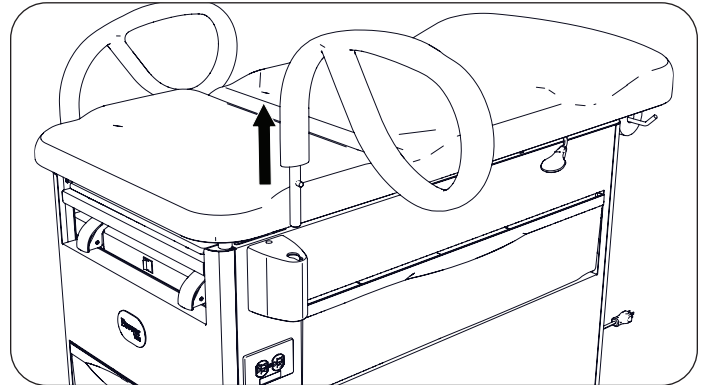


- Using supplied pry bar, remove three push rivets attaching handle support cover, one on top and two on bottom of cover. Remove cover.

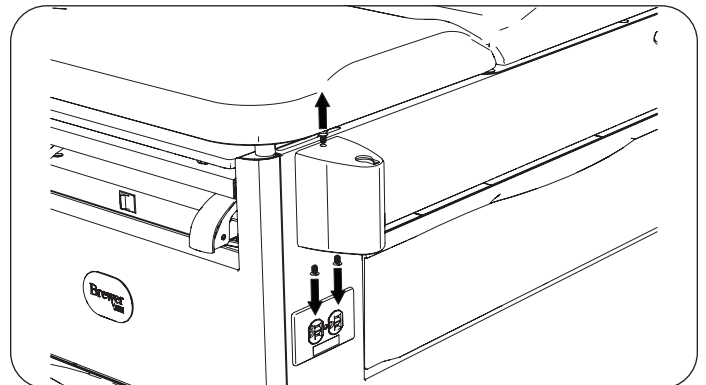


- Remove four hex head bolts attaching handle support to mounting plate. Remove handle support.

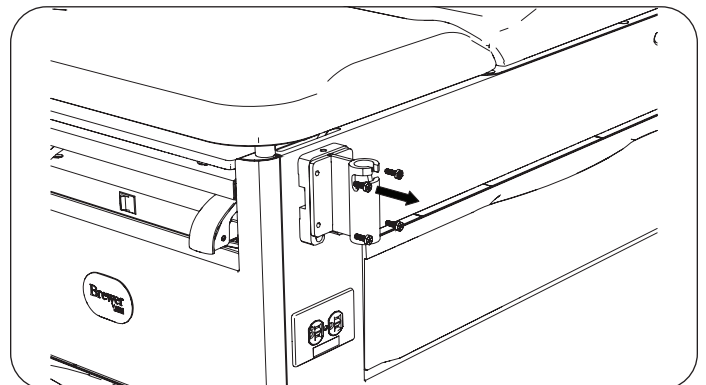

4. To gain access to inside of cabinet, pull out stirrups (if equipped) and leg extension. Remove upholstered pad and debris pan.

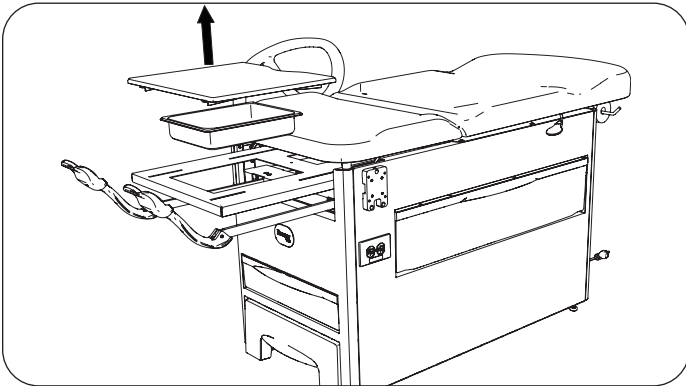


5. Remove mounting plate by loosening three hex head bolts from inside of cabinet. Discard mounting plate.

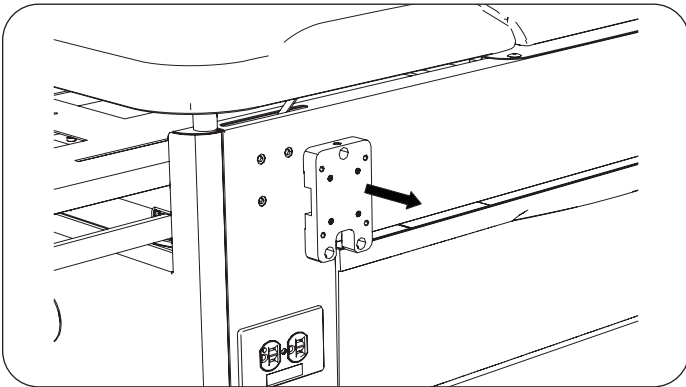


6. Position new mounting plate and thread/tighten three hex head screws into new mounting plate.


Note: New mounting plate may have a plunger installed which does not affect grab bar operation.

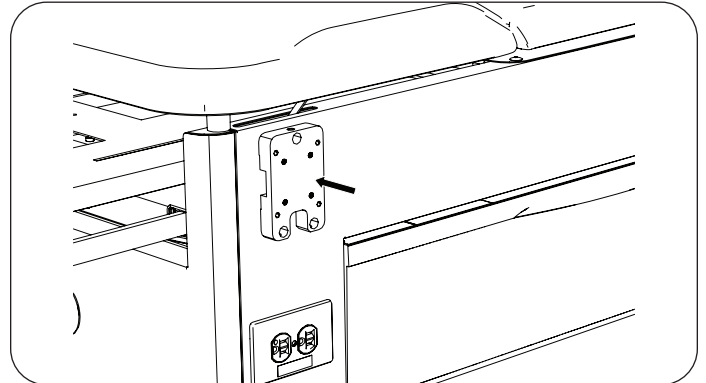


7. Attach handle support to mounting plate using four hex head bolts.

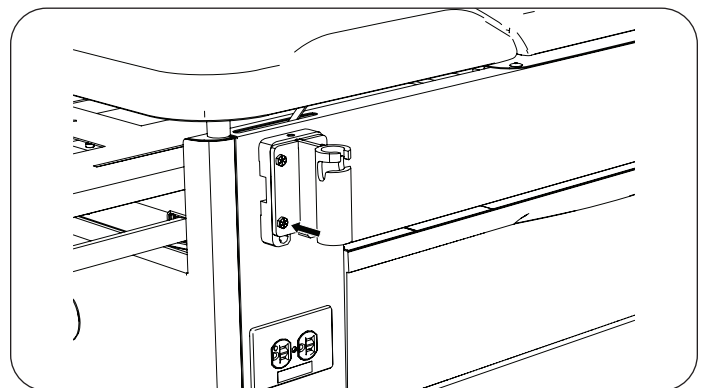

8. Attach handle support cover to handle support using three large push rivets, one on top and two on bottom of cover.

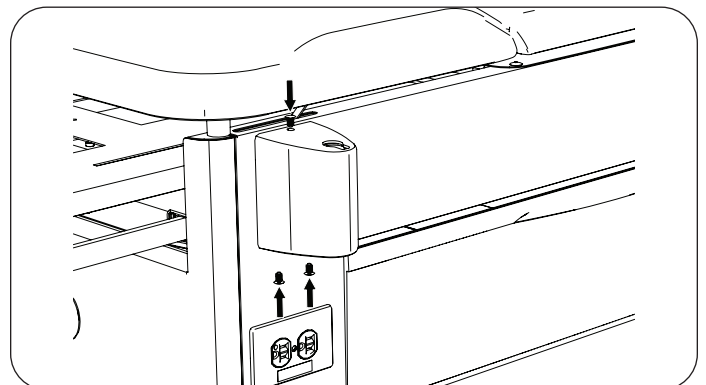

9. Install assist handle.

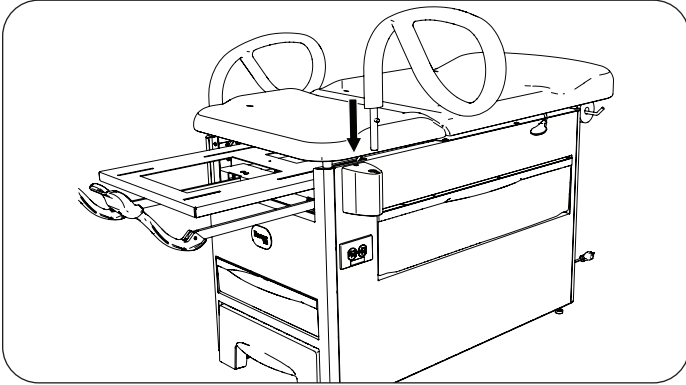


10. Repeat for opposite side of table.


11. Install upholstered pad and debris pan. Push stirrups (if equipped) and leg extension back into table.

With Assist Handles and Knee Crutches


1. Remove assist handle.

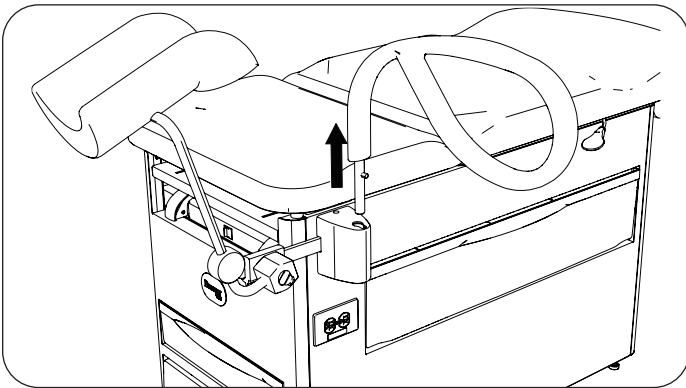


2. Remove knee crutch.

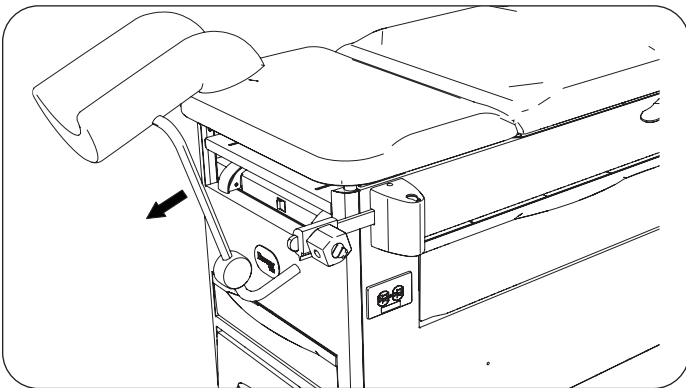


3. Remove knee crutch lock socket.

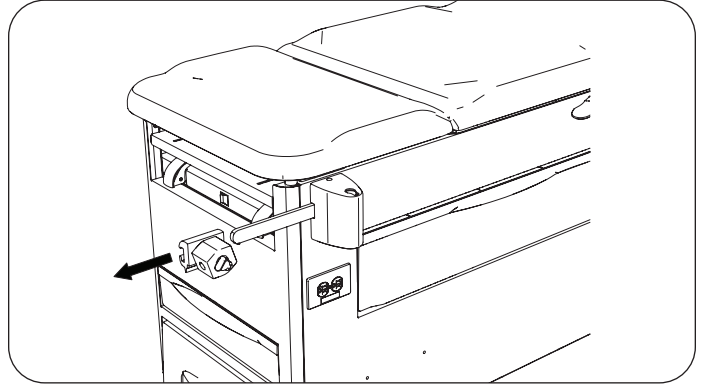


4. Using supplied pry bar, remove three push rivets attaching handle support cover, one on top and two on bottom of cover. Remove cover.

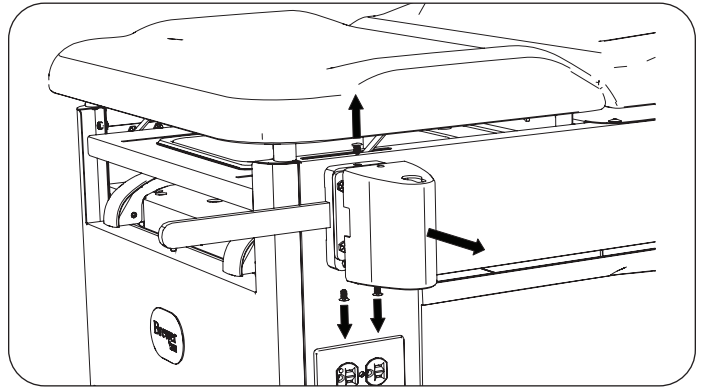


5. Remove four hex head bolts attaching handle support to mounting plate. Remove handle support.

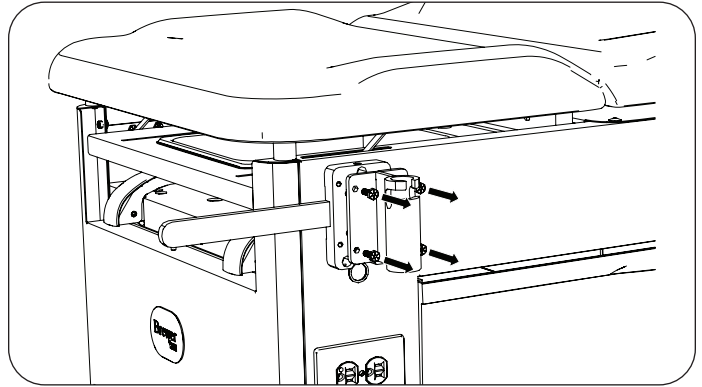


6. To gain access to inside of cabinet, pull out stirrups (if equipped) and leg extension. Remove upholstered pad and debris pan.

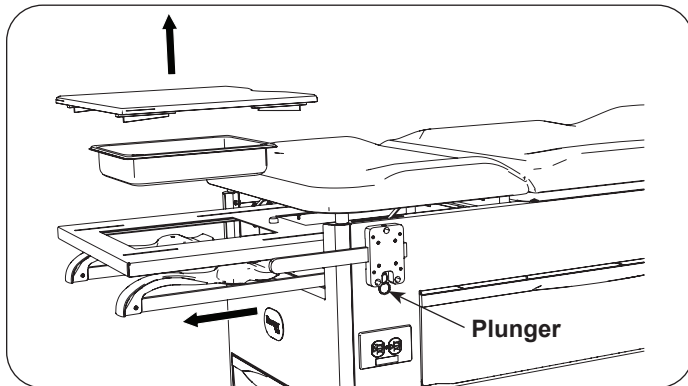


7. Remove mounting plate and beam by loosening three hex head bolts from inside of cabinet. Pull plunger downward and remove beam from mounting plate. Discard mounting plate with plunger.

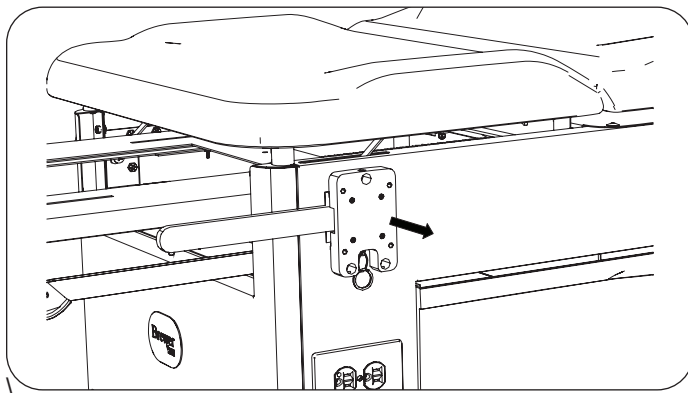


8. Pull plunger downward and install beam in new mounting plate. Round end of beam faces forward and set screws face downward. Position new mounting plate and thread/tighten three hex head screws into mounting plate.

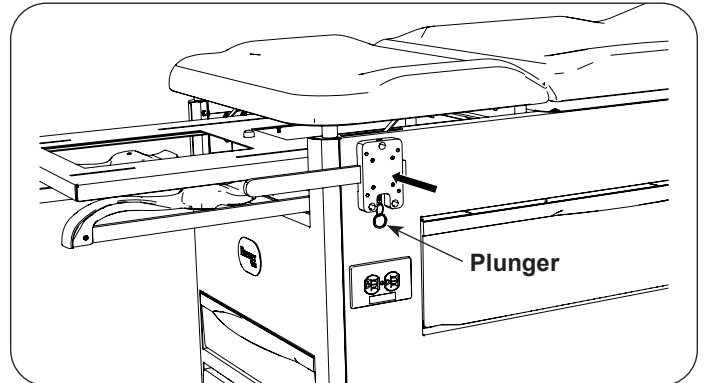


9. Attach handle support to mounting plate using four hex head bolts.

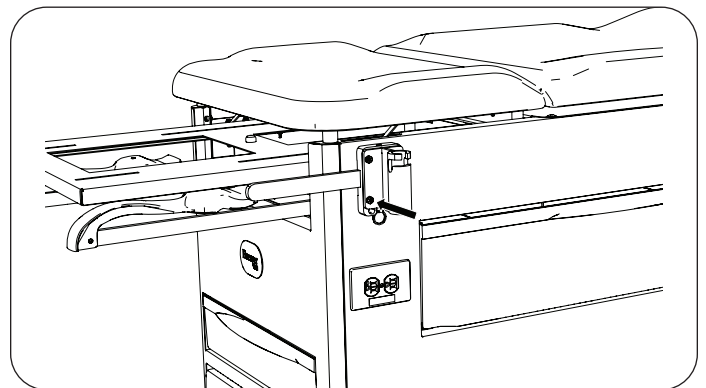


10. Attach handle support cover to handle support using three large push rivets, one on top and two on bottom of cover.

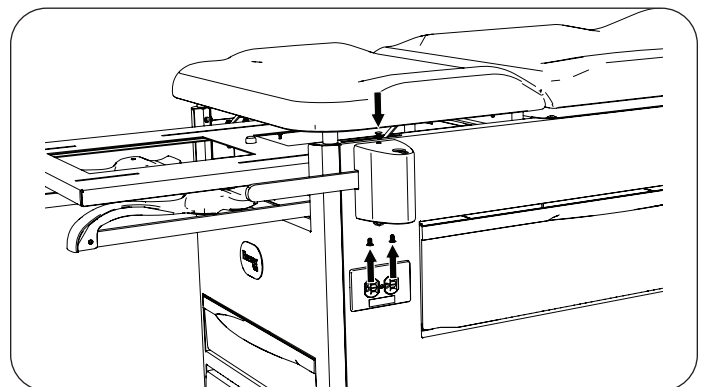


11. Install knee crutch lock socket.

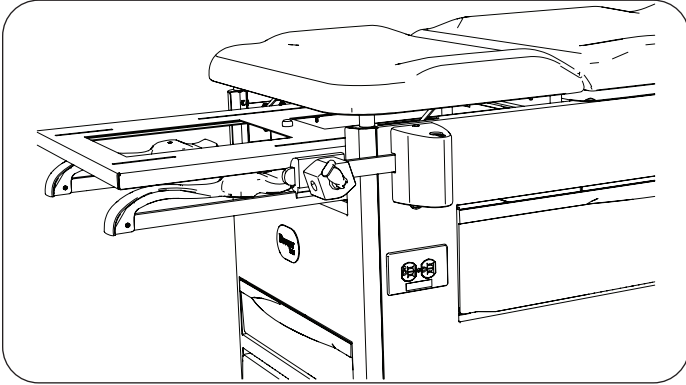

12. Install knee crutch.

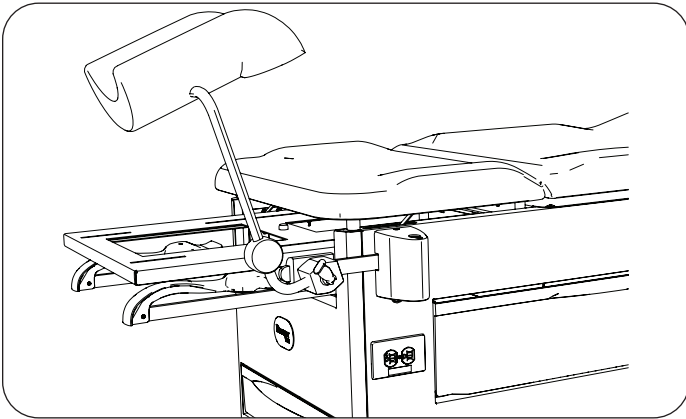

13. Install assist handle.

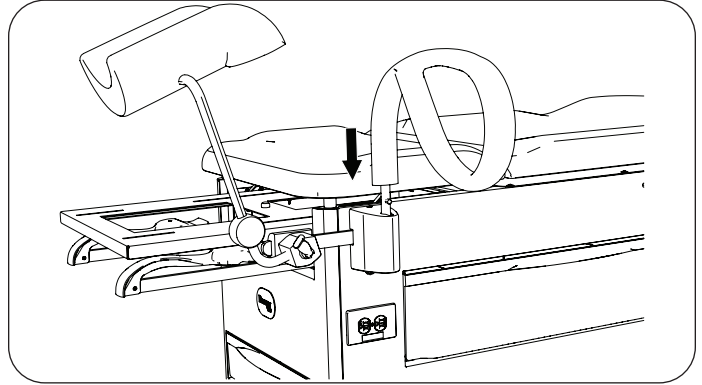

14. Repeat for opposite side of table.

15. Install upholstered pad and debris pan. Push stirrups (if equipped) and leg extension back into table.